



# STERICLIN® LIGHT BLUE TINT ROLLS - FLAT

## ISO 11607- 2 EN 868-3 and -5 for laminates

### STERICLIN® FLAT ROLLS

ISO 11607- 2 ; EN 868-3 and -5 for laminates

**Construction:** 70 g/m<sup>2</sup> sterilization paper conforming to EN 868-3 and DIN 58953, parts 2 and 6.

- light blue tinted transparent film Polyester / Polypropylene (PETP/PP).
- heat sealable, sterilization resistant colours, water based.
- non-toxic STEAM, E.T.O.-GAS and FORMALDEHYDE indicators at intervals of 135 mm

### Special features

i/d of core 76 mm, inside core QC-label with batch-Number etc.

**Printing details:** Indicators, description / size, article Number and batch Number after the backslash, manufacturer and brand name are printed onto the paper, covered by the seal.

**Method of sterilization:** Steam, E.T.O.-gas, formaldehyde - not for gamma-rays or plasma-Sterilization!

**Sealing:** For see-through reels commercial continuous sealing devices are most suitable. The sealing temperature should be in the range of 180°-200°C.

**Sizes:** Flat reels are available in 9 standard widths, length 200m.  
Method of use

### Method of use

STERICLIN® see-through reels should be cut off, according to the item to be packed. Make a bottom seal right at the end of the tube. Fill the pouch and seal at the top end leaving a peel tab of 15 to 20mm.

A free space of at least 3 cm from the seal area should be maintained in order to avoid stress on the seal. To remove content peel apart the film and the paper.

STERICLIN® reels are in accordance with DIN EN 868-5. VP STERICLIN® is certified to EN ISO 9001. The CE-kitemark is printed onto the outer carton.

ARTG No. 135955

Reels – printed, with Indicator

Light Blue Tint Rolls – Flat		
Code	Description	Ctn / Qty
800301	50mm x 200 metres	1 Roll
800302	75mm x 200 metres	1 Roll
800303	100mm x 200 metres	1 Roll
800304	150mm x 200 metres	1 Roll
800305	200mm x 200 metres	1 Roll
800306	250mm x 200 metres	1 Roll
800307	300mm x 200 metres	1 Roll
800308	380mm x 200 metres	1 Roll
800409	420mm x 200metres	1 Roll

Distributed by



Australian Head Office

PO BOX 465  
Blackburn, VIC 3130

Free Call: 1800 723 405  
Ph: (03) 9845 0300  
Fax: + 61 3 9845 0350  
e-mail: [sales@pacificlab.com.au](mailto:sales@pacificlab.com.au)

Connect with us:

 /PacificLab

 @PacificLabP

